Neoflex Fibuvit 390

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

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Status: 04/2015

Specification

The anatomically shaped thigh support Neoflex Fibuvit 390 provides secure support and warmth to musculature. The upper width can be adjusted individually by Velcro. The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic Neoprene provides high wearing comfort.

Indication

- lesion of lower leg musculature (muscle fiber rupture contusion)
- enthesopathy and myalgia
- prophylaxis during sport activities

Contraindication

Unknown.

Instructions

Pull the support over the afflicted lower leg and place the reinforced seam at the back of the leg. Adjust the upper width individually by Velcro. For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Neoflex Fibuvit 390. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Washing instruction (30°)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports the blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

Color

black