
Manuvit Comp 630

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

Web: www.orpedo.com
Mail : info@orpedo.com

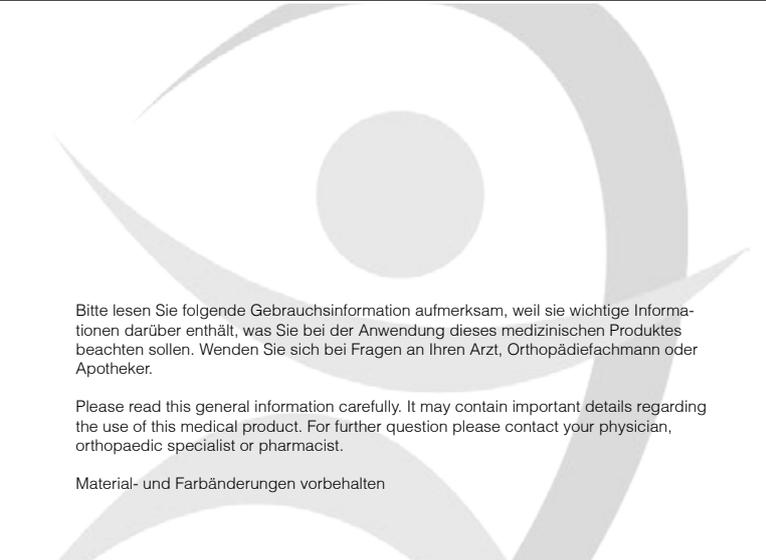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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 04/2015

Product description

The Manuvit Comp 630 wrist brace is anatomically knitted out of two-ply elastic material. A flat knitting technique is used in its manufacture. The brace has integrated silicone pads and a plastic splint, used to optimally position the wrist. It provides support, and assists the natural movement of the wrist. The thumb area is made of elastic fabric, for added support comfort. The breathable material is particularly skin-friendly, and, due to the good climatic characteristics, it is comfortable to wear. The brace adapts to the body's movements. Constriction and slippage is prevented. The strap is used to individually set the compression level.

Indications

- Carpal tunnel syndrome,
- Tendovaginitis (tendonitis)
- Distortions
- Contusions
- After cast removal
- Degenerative changes (arthrosis/arthritis)
- Inflammation
- Trauma
- Postoperative situations
- Dislocations / lesions / sprains

Contraindication

Unknown.

Use instructions

To put on the Manuvit Comp 630 brace, grasp the upper end and pull over the wrist. The affected joint must be completely covered by the profile pads. After the affected hand is placed in the bandage, close it with the strap.

Material composition

Polyamid: 63% Spandex: 15%
Cotton: 12% Polyester: 10%

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swelling, allergies and other diseases or complication. The first application an adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the manuvit comp 630. In case if numbness or coldness loosen the support or take it off at once. This product has its full effect on the movement. For this reason the bandage has to be taken off in case of long respites or while sleeping. Ask your physician or orthopaedic specialist or pharmacist to check the fit and adjustment.

Washing instruction (30°)

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Do not use fabric softener because this affect the compression quality. Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry.