## Umerovit-Air 360

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 05/2015

## Specification

The shoulder abduction pad „Umerovit 360“ consist of a breathable, resistant and light material. Due to the individual filling of the air bag, different abduction grades are enabled. The air cushion will be fixed by setting the belt at / by the concerned arm. The ring belt will unload the shoulder-neck-area. The orthosis will relieve the subacromial space and avoid adhesions in the area of the subacromial, subcoracoid, and subdeltoidal. The risk of a re-dislocation after an anterior shoulder dislocation will be considerably reduced. A manual pump will be included in the dispatch.

## Indikations

- Impingementsyndrom
- frozen shoulder
- mobilization and rehabilitation after an operational measure
- shoulder dislocation
- relieving of the subacromial space
- avoiding of adhesions in the field of the subacromial, subcoracoid and subdeltoidal area

## Contraindication

Unknown.

## Instructions

The Umerovit 360 will be fixed with the belts at the affected arm. The belts are provided with Velcro closures which enable an easy handling. The correct positioning of the arm have to be observed. The product is to be used following the instructions and purposes.

## Notes

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complications. The first application and adjustment should be done by your physician or orthopaedic specialist to ensure the correct fitting of the splint. Should appear a sensation of cold or numbness in your arm or shoulder, please relax or put down the shoulder abduction pad. The size and the fitting should be checked by a physician or specialist. Please get in contact with your physician, pharmacist or orthopaedist.

## Washing instruction (30° )

The pad should be handwashed in a mild detergent at a temperature of 30°C (86° F). Do not use fabric softener. Avoid direct sunlight, heat of radiators, electric iron or similar. Do not bleach or dry clean.

## Color

black