
## Epivit Comp 620

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

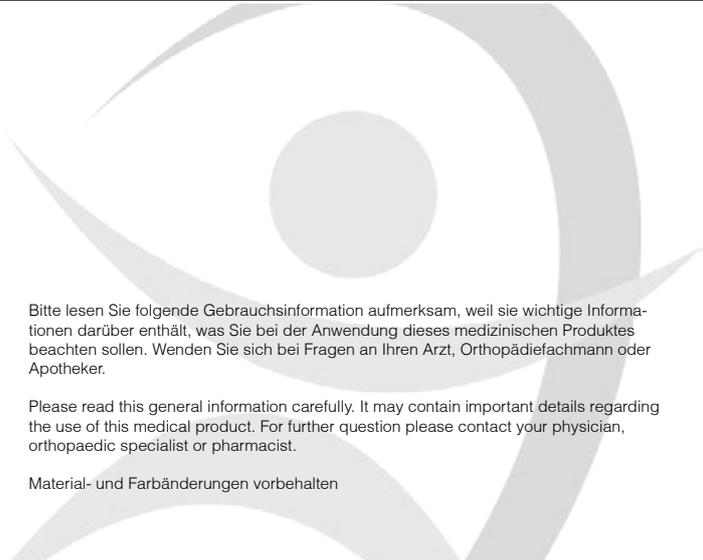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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 05/2015

## Product description

The elbow bandage Epivot Comp 620 is fitted on the inside with two insole pads which relieve the muscle insertions in connection with the anatomically constructed stitching material by a raised compressing effect. By means of the medically indicated different compression areas and the optimal selection of material, the bandage has a stable position and achieved excellent climatic values during the wear test. In motion, the active bandage causes a pain relief in combination with the viscoelastic insole pads, it helps to relax tensions and activates the muscle regulation by an improved proprioception.

## Indications

- enthesopathy, epicondylitis in the elbow area (tennis elbow or golf elbow)
- after trauma and slight injuries
- inflammation
- prophylaxis at sportive activities
- bursitis (inflammation of an elbow bursa)
- postoperative and for after-treatment of heavy injuries
- degenerative alterations of the articulation of the elbow
- arthrosis / arthritis
- overburden symptoms

## Contraindication

Unknown.

## Instructions

Pull the Epivot Comp 620 over the arm until the insole pads are sitting on the muscle insertions, a little bit under the elbow center. Because of movement of the soft pads, a reapplication might be necessary. Use the product according to the indications and the purpose.

## Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Epivot Comp 620. In case of numbness or coldness loosen the support or take it off at once. This product has its full effect on the movement. For this reason the bandage has to be taken off in case of long respites or while sleeping. Ask your physician or orthopaedic specialist to check the fit and adjustment.

## Washing instruction (30° ☞ ☒ ☒ ☒)

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Do not use fabric softener because this affects the compression quality. Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

## Color

black-white, skin