
## Humerovit 350

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

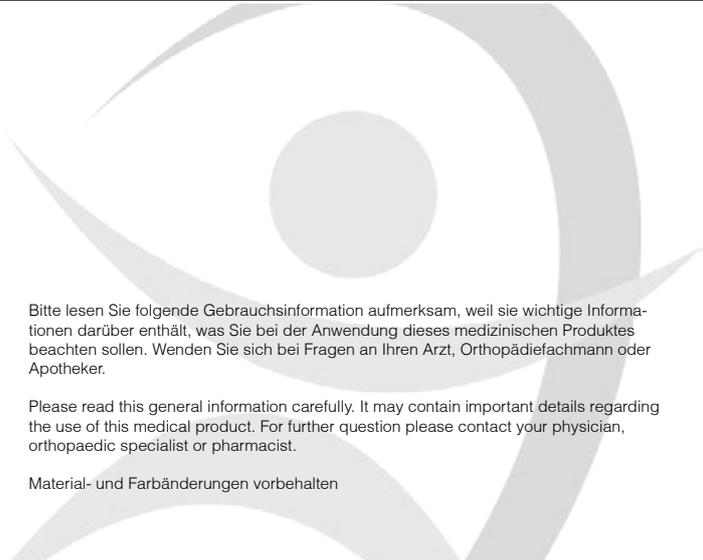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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 05/2015

## Product description

Shoulder arm orthosis Humerovit 350 to immobilize the shoulder joint resp. the acromioclavicular articulation. The positioning of the arm is adjustable individually. In case of injuries in the cranial humerus area resp. in the acromioclavicular articulation the horizontal, in circular manner taut positioning belt provides an optimized position of the head of the humerus. By this, an excellent immobilization of the complete shoulder joint is achieved. The shoulder cap is stabilized by help of an additional Velcro to keep forward resp. backward movements as low as possible. For a better positioning of the hand there is an additional Velcro. The optimized positioning of the complete arm offers a wide range of use for the following indications. According to the indication the circular positioning belt should be fixed at the correspondent height around the humerus.

## Indications

- Injuries of the rotator cuff, insufficiency of the capsular and ligament apparatus
- Immobilization of acromioclavicular joint after trauma.
- For postoperative care of radial and/or ulnar fracture
- Supcapitale fracture of capitulum of humerus
- for post-traumatic or postoperative stabilization
- distortion, luxation
- traumatic injuries, lesions
- bursitis olecranon

## Contraindication

Unknown.

## Instructions

The healthy arm is lead through the slit in the orthosis. The longer part of the orthosis is lead behind the head with the shoulder cap over the affected shoulder. Now put the elbow into the second cap and wind the positioning belt over the other end in circular manner around the body, between the body and the affected upper arm forward and close it around the arm in backward direction at the exterior side. Close the other end of the orthosis under the wrist upwards. For the application of this product you have to follow its description andintention.

## Notice

Please contact your physician or pharmacist in any of the following cases: Severe injuries, open wounds, swellings, allergies and other diseases or complications. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Humerovit 350

## Material

Polyurethan  
Nylon  
Polyester

## Washing instruction (30°

The orthosis may be washed by hand in mild detergent at a temperature of 30° C(86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

## Color

black