
Thorax 910 (Damen)

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

Web: www.orpedo.com
Mail : info@orpedo.com

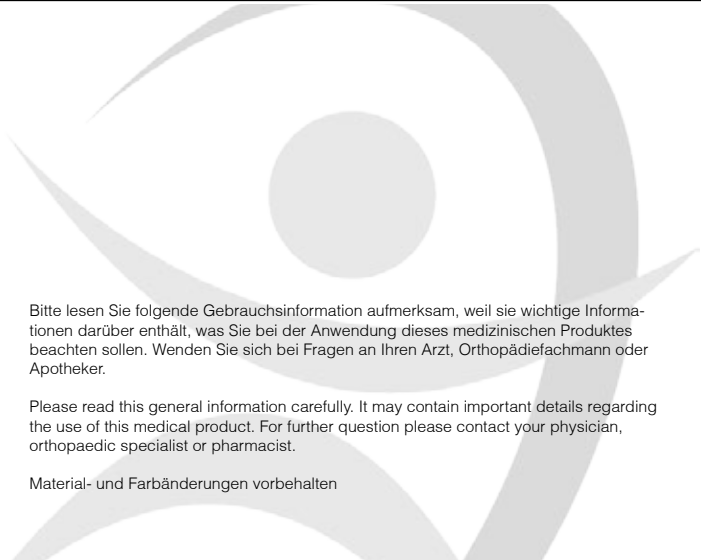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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 04/2015

Specification

The Thorax 910 (Woman) reduces the movement of breathing of the chest by circular compression in the area of the thoracic cage. The healing of contusion and other traumas is supported.

Indication

- distortion of thoracic vertebral column
- rib contusion
- rib fracture
- inflammation of ribs and spine
- degenerative deformans
- postoperative

Contraindication

Unknown.

Instructions

Place the support round the ribs and close in front with Velcros. For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: Severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Thorax 910 (Woman). Do not wear the support while sleeping. Do not use ointment or creams while wearing the support.

Washing instruction (30°)

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

white